VISMARA  
BEAUTY & SPA DESIGN

Edge

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
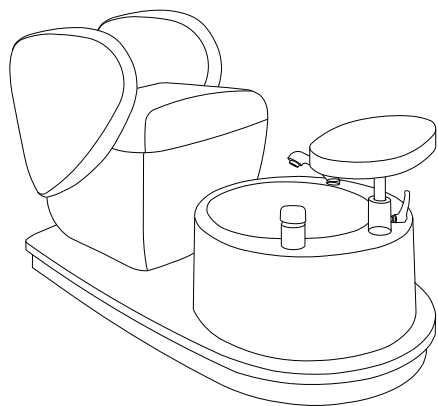
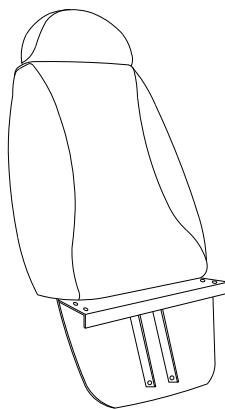
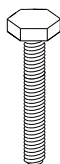
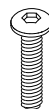
# Dimensioni / Dimensions

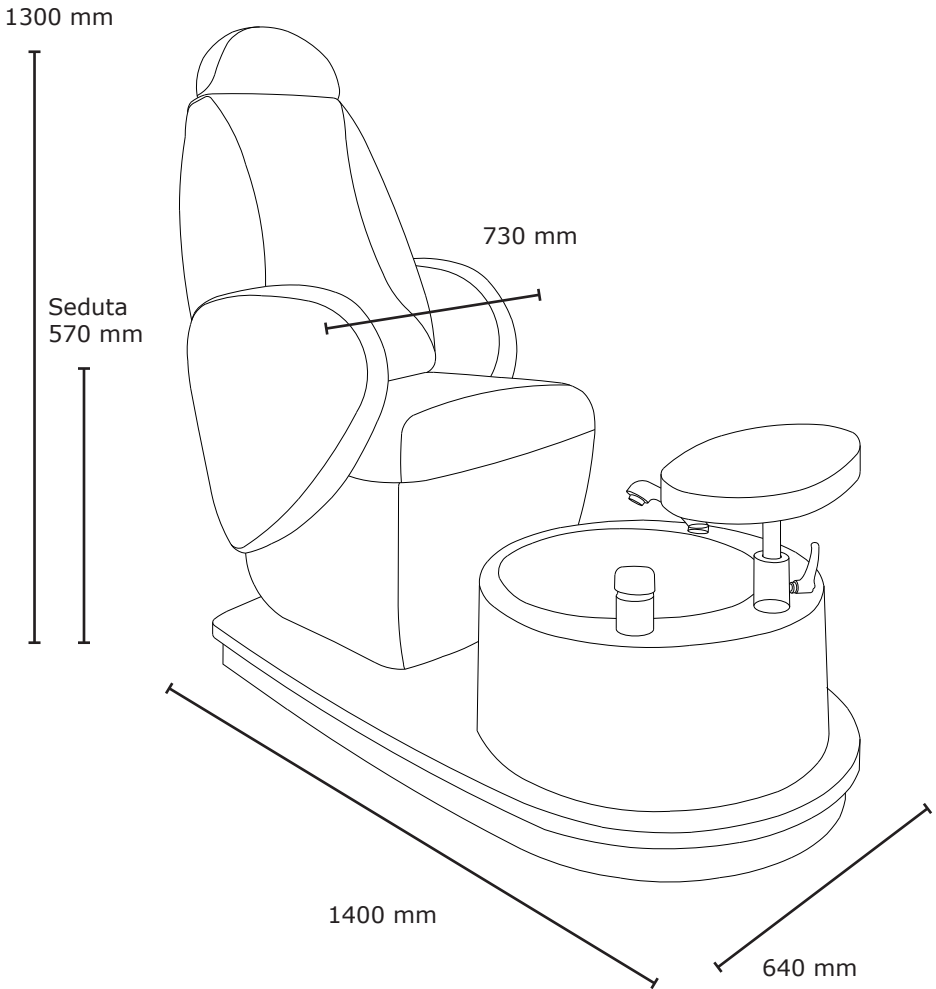---

## Collegamento idraulico / *Water system connection*

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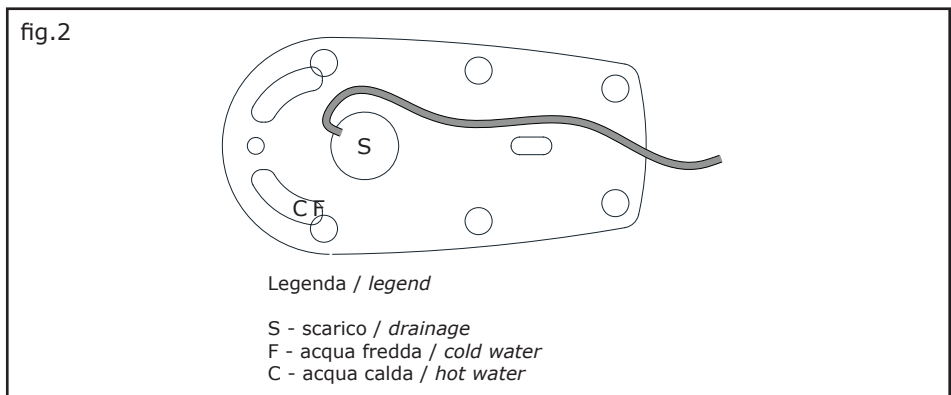
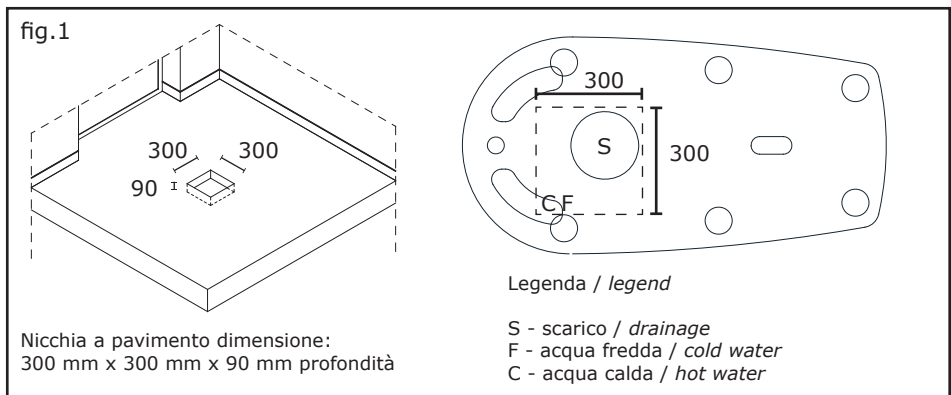
*The connection of the Podiatric chair Saint Barth has to be executed by qualified professionals, who should issue a certification of their work, and has to follow the scheme below.*

*The drain connection can be done under the floor, when a special floor niche is provided. In this case connect the drain to the outlet of the siphon placed downwards, with a diameter of 40 mm (see fig.1).*

*Alternatively, drainage can be connected to the floor, using a flexible drain pipe. In this case, connect the drain to the outlet placed horizontally, with a diameter of 32 mm (see fig.2).*

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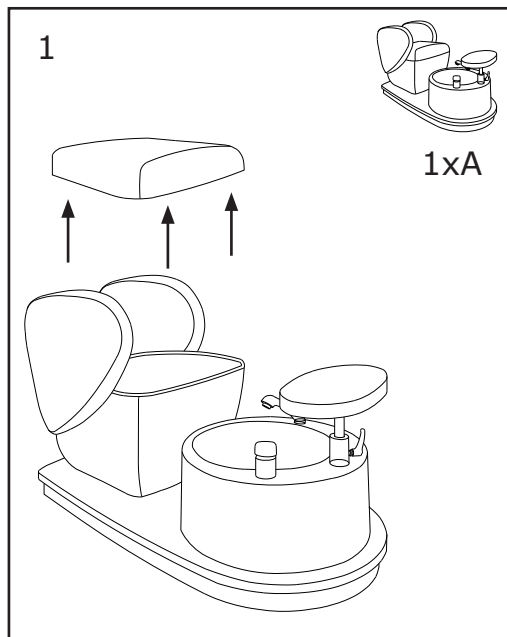
*Semi-rigid or rigid piping, for 40mm diameter underfloor solution, for a 32mm diameter floor solution. It is advisable to guarantee a slope of 1.2 cm for each linear meter to ensure adequate drainage.*

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
*Place the base (A) and lift the seat of the chair to allow the fixing of the back (B).*

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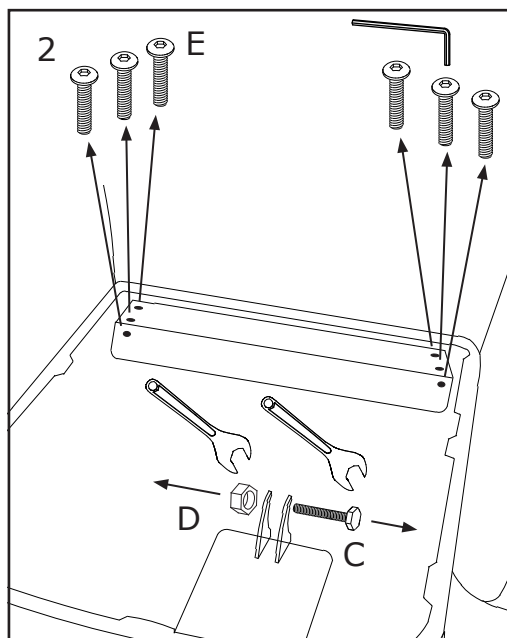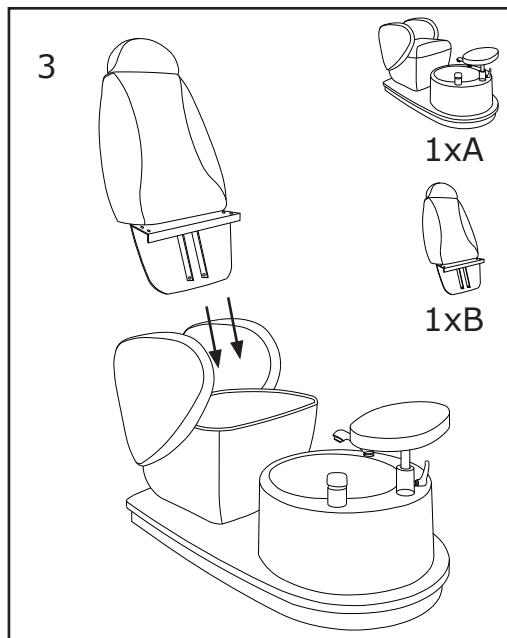
*Unscrew and remove the pre-tightened screws from the 6 holes provided for fixing. Unscrew the bolt and remove the screw from the central pin, through the use of 2 wrenches no.13.*

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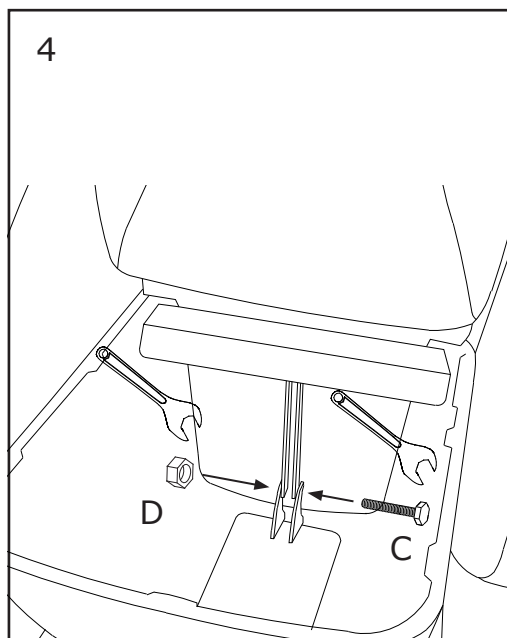
*Insert the back (B) into the appropriate housing, matching the screw holes and the central pin.*

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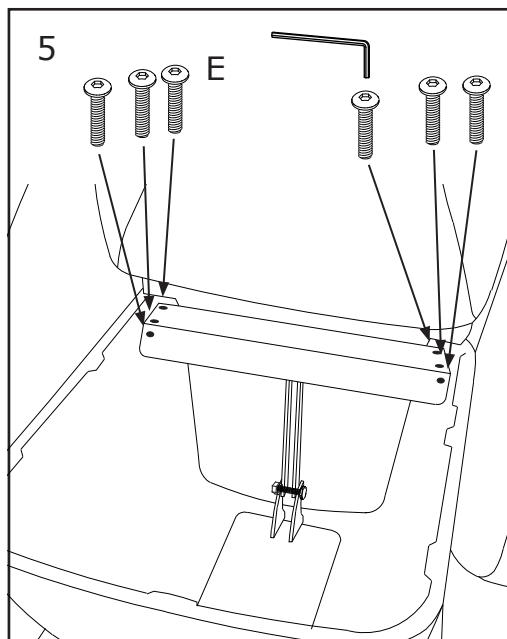
*Fasten the central pin to the base using a threaded screw and tighten the self-locking bolt using 2 wrenches no.13.*

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
*Fasten the metal part of the backrest to the base support, as shown in the figure, using an Allen key.*

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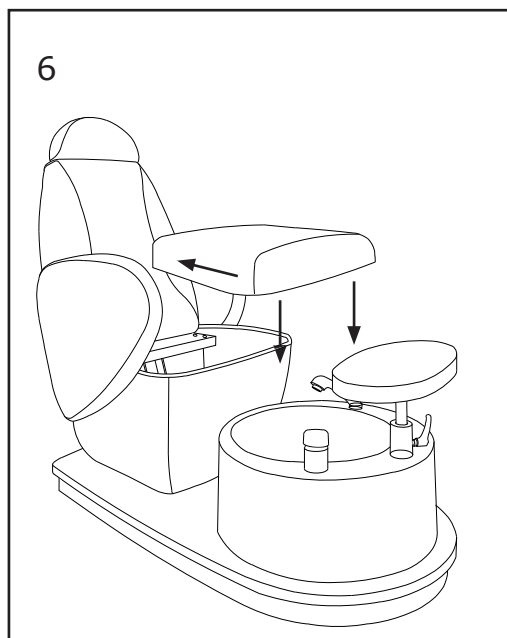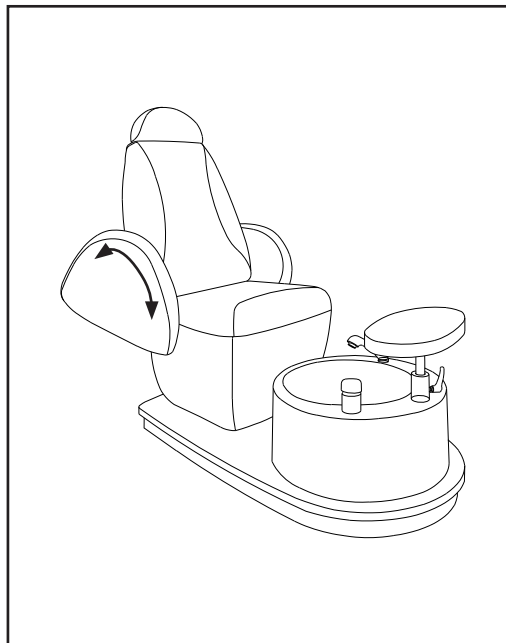
*Reposition the seat of the chair by pushing it towards the back and lowering it, making it collimate with the base of the seat.*

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
## Arm

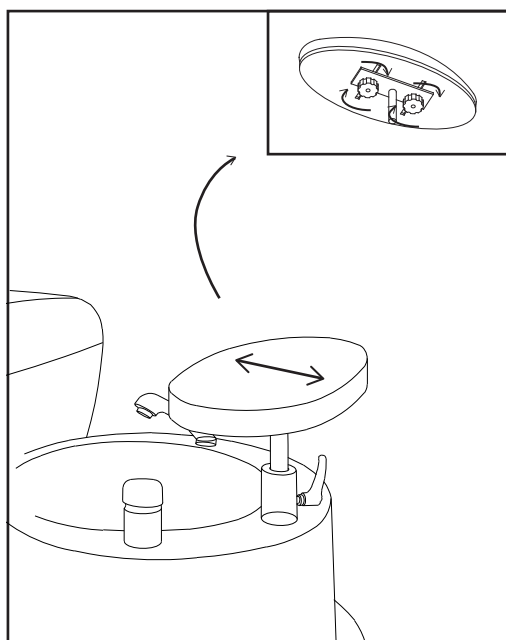*Raise the armrests by rotating them upwards to facilitate sitting.*

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
## Horizontal regulation (footrest)

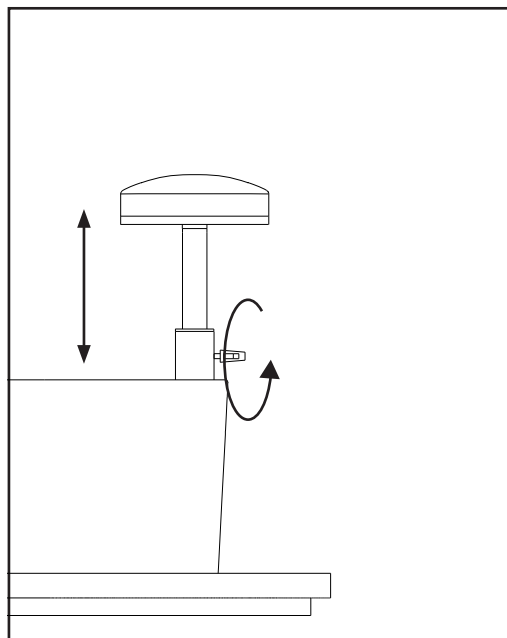*Horizontal regulation of the position of the footrest by using the two dedicated control knobs placed beneath the footrest.*

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## *Height regulation (footrest)*

*Height regulation of the footrest by the rotation of the dedicated control knob.*

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---

*The multifunctional podiatric chair Edge ensures an excellent comfort to both the customer and the operator, while its simple management in its movements and regulations allow the operator to carry out his work even in narrow places.*

*The chair's functions are adaptable to anybody, and its hypoallergenic padding is entirely washable.*

*The cleaning of the basin is fast thanks to the shower hose, placed at the base of the chair, which allows a quick sanitation.*

#### **BEFORE EACH TREATMENT**

*Fill in the basin minding the water level.*

*Check that the water temperature does not exceed 38°C.*

#### **AFTER EACH TREATMENT**

*Basin cleaning: use common liquid neutral detergent such as soap and a damp cloth  
Do not use ammonia or acetone based cleaning products since their use would permanently damage the chair.*

*Also, do not use formic acid or formaldehyde based products for the sanitation of the basin.*

*For any inquiry please contact your vendor.*

*VISMARA declines any responsibility for any damage caused by negligence regarding what mentioned above.*

*Seat cleaning: In order to clean and sanitise the chair after each treatment use a common liquid neutral detergent such as soap and a damp cloth*

*Do not use ammonia or acetone based cleaning products since their use would permanently damage the chair.*

#### **PERIODIC MAINTENANCE**

*It is advisable, for the best chair functioning to have it maintained periodically by your vendor.*

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---

*The installation has to be carried out by qualified staff, able to certify their work.  
Do not proceed with any treatment to patients with inflamed legs and feet, with open wound, infections or continuous pains.*

*In case of even light sickness of the patient stop the treatment immediately.*

*Read the present manual carefully before operating the chair. The use of the podiatric chair is for qualified personnel only.*

*The podiatric chair has to be connected to an electric system in compliance with the  
In case these requirements are not met the use of the chair is forbidden.*

*Before any treatment please ensure that every part of the chair is intact and that it does not present any sign of wear: if that is not the case stop the treatment immediately.*

*Do not use solvents or chemical additives for the cleaning of the chair.*

*Do not handle flammable liquids nearby the chair.*

*In case of water leaks, unplug the chair immediately.*

*Before any cleaning or maintenance operation unplug the chair.*

*Unplug the chair when not in use.*

*Keep children away from the chair.*

*Do not touch or tamper with the electric and hydraulic components.*

*Each part of the chair has to be correctly installed.*

*Do not trash the packaging.*

*The company declines any responsibility for fire or damages to people and things.*

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*In case of malfunction please contact your vendor only.*

## Problemi e rimedi / Problems and remedies

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*In case of malfunction, do not touch or tamper with any hydraulic component. Please contact your vendor only.*

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*The right of call and the work hours, to be settled directly with the customer support, are at expenses of the inquirer, for any intervention requested without actual deficiency of the product.*

*The present warranty does not cover the parts of the chair that are subject to wearing, such as wooden parts and any other components that will turn out defective as a result of negligence, carelessness in the use, wrong installation or maintenance, intervention by unauthorized personnel, careless shipping.*

*The manufacturer declines any responsibility for any direct or indirect damage sustained by people, animals and things.*

*The present warranty will not be lengthened in case of proven malfunction.*

---

---